


SHEEP IMMOBILIZER

INSTRUCTIONS FOR USE

1

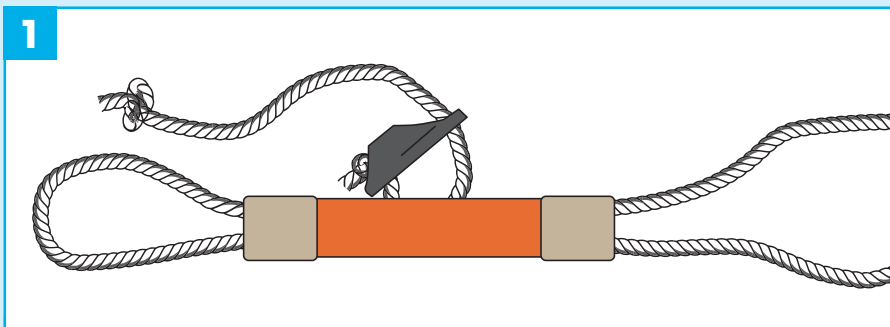


STEP 1

When placing the sheep immobilizer on the sheep ensure that the black plastic rope clamp is facing up and is pulled up firmly into the pipe. Place the first front leg in the loop on the same side as the rope clamp.


STEP 2

Pull the rope through to make the next loop. Always keep the rope tight around the last leg, while placing the next leg in the next loop.

2

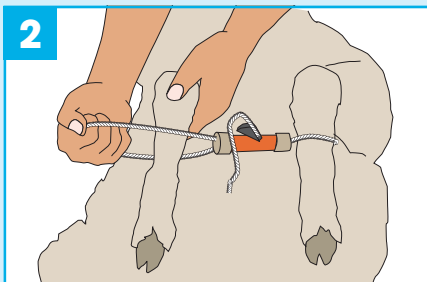

3

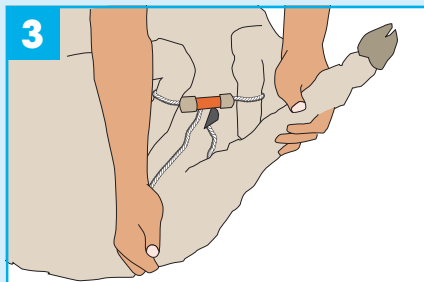


STEP 3

Pull the rope through to make the third loop, and place the hind leg in the loop over the hamstring.


STEP 4

To tighten, hold the rope clamp with one hand and pull the end of the rope as tight as possible with the other. Place the rope firmly in the rope clamp. Hold the orange pipe and push the hind leg down to lock the rope tight in the clamp.

4

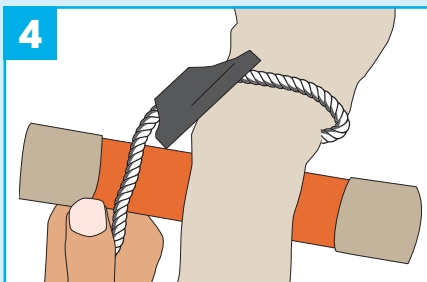

5

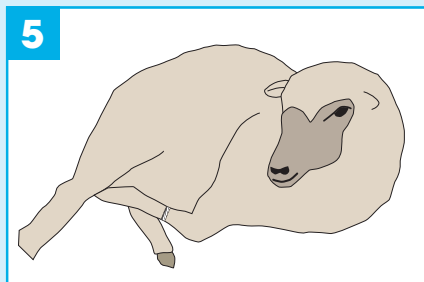

STEP 5

Lay the sheep on its side with the free hind leg facing upwards. To remove the immobilizer pull the tail of the rope out of the clamp and release the legs.

HEAD OFFICE AUSTRALIA

Leader Products Pty Ltd
465 Hume Highway
Craigieburn VIC 3064
Australia

Postal address
PO Box 42
Craigieburn VIC 3064
Australia

General enquiry emails
enquiries@leaderproducts.com.au
Sales emails
sales@leaderproducts.com.au

STATE BRANCHES

For full contact details please refer to our website: www.leaderproducts.com.au

VIC/TAS 03 9308 2355
NSW/ACT 02 6882 7995
SA 08 8391 2011
QLD/NT 07 4633 1662
WA 08 9277 5150
International +61 3 9308 2355