

HEAD OFFICE AUSTRALIA

Leader Products Pty Ltd
465 Hume Highway
Craigieburn VIC 3064
Australia

Postal address
PO Box 42
Craigieburn VIC 3064
Australia

General enquiry emails
enquiries@leaderproducts.com.au
Sales emails
sales@leaderproducts.com.au



STATE BRANCHES

For full contact details please
refer to our website:
www.leaderproducts.com.au

VIC/TAS 03 9308 2355
NSW/ACT 02 6882 7995
SA 08 8391 2011

QLD/NT 07 4633 1662
WA 08 9277 5150
International +61 3 9308 2355

BOLUS APPLICATOR

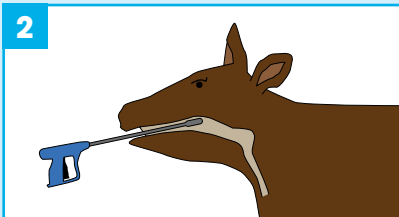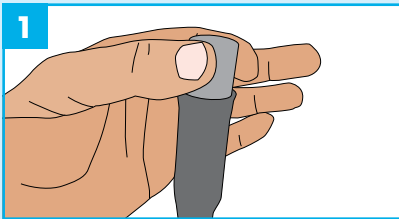

HOW TO APPLY

STEP 1 Remove the bolus from the canister and insert it completely into the retaining cup until the push rod is engaged.

STEP 2 Place the applicator into the centre of the animal's mouth just beyond the back of the tongue and depress the trigger. If there is resistance, adjust the position of the applicator until the bolus releases easily.

If swallowing reflex does not occur easily, check that the animal's head and the applicator are straight and that the trigger is vertically aligned.

